NL

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EN

Directions for use:

- In rare cases, the use of CrossLinq can cause skin irritations or allergic reactions. The treatment should be stopped immediately if adverse reactions occur and the tape should be removed from the skin.

- It is recommended that the user first applies a small piece of tape if he/she is over-sensitive to the use of plasters or tape. It must then be borne in mind that allergic reactions do not always occur immediately, but that a later reaction (after approximately 24-48 hours) can also occur. Remove the product immediately in case of adverse reactions.
- Do not use CrossLinq on wounds, burns and mucous membranes, nor on the mouth, nose or eyes.
- The skin should be cleaned before applying the CrossLinq.
- When removing the CrossLinq, the tape should be removed slowly in the direction of the hair growth. If the removal of the tape proves difficult, it is advised to use a tape removal agent.

Instructions

- Keep CrossLinq at room temperature (between 4 – 25 °C) and not in direct sunlight.
- Do not put CrossLinq on or near radiators or any other source of direct heat.
- Keep the tape out of reach of children.
- For single use only.
- Injuries can always occur, even when CrossLinq is used in the proper manner.

THYSOL Group BV are not liable for any injuries and/or damage due to CrossLinq or the unprofessional use of the product.

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