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Wydanie 20.12.2021
Edition 20.12.2021
Ausgabe 20.12.2021

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EN

Indications:

Stabilisation of the thoracic-lumbar-sacral spine (From Th6-L-S) in acute, chronic, as well as non-operative conditions, deviation of the vertebral body, immobilisation following the neurosurgical, neuro-orthopaedic, oncological surgeries and other.

⚠ Contraindications:

Unstable vertebral fractures and dislocations requiring surgical treatment. Considerable degree of vertebral injury of inflammatory and neoplastic aetiology. Lesions causing deformation of the vertebrae and nucleus pulposus prolapse requiring another form of immobilisation. Conditions after spinal surgeries, especially the ones involving bone grafting.

Description of the product:

Light aluminium construction enabling perfect adjustment of the frame to the patient's body. Soft padding ensures comfort of use. Both the height and the width of the upper and the lower pelotte can be adjusted. User-friendly and comfortable fastening of the posterior pelotte enabling the regulation of its back pressure.

Manner of regulating and adjusting the corset:

The corset should be selected, adjusted and put on by qualified health care professionals. Please choose the appropriate size of the corset basing on the size table presented below.

Screws enable the individual adjustment of both the height and the width of the corset. Unscrewing these screws allows to adjust these parameters. The height and the width should be adjusted symmetrically on both side of the pelotte. The position of the posterior (stabilising) pelotte should remain symmetrical. All adjustments should be performed before supplying the patient. The corset should be worn on a cotton T-shirt.

Put the corset on when the patient is lying down. The frame of the corset should be placed in a way ensuring that the sternum pelotte rests on the manubrium sterni, whereas the lower pelotte rests on the lower abdomen (above the upper edge of pubic symphysis). The position of the posterior pelotte has to be adjusted to the height of the injury, and it has to meet the medical recommendations. The length of the belts should be properly adjusted, in order to ensure proper adhesion of the posterior pelotte – appropriate stabilisation.

Maintenance:

Elements made of eco-leather should be cleaned with a damp cloth and with the use of gentle washing agents (f. ex. soap). Do not use strong detergents and bleaching agents.

**Size:**

Size*	S	M	L	XL
Measurement between the manubrium sterni to pubic symphysis (cm)	37-47	40-50	43-53	45-56
Waist circumference (cm)	60-75	75-90	90-105	105-115

* The selection of the size should be made basing on the measurement of the distance between the manubrium sterni to pubic symphysis. The waist circumference stands as an auxiliary measurement.

⚠ NOTE:

Failure to follow proper personal hygiene, using inappropriate laundry detergents may result in abrasions, as well as other conditions related with excessive sweating and the development of bacterial flora. Do not use the orthosis on gels and warming ointments. Remember that unrinsed residues of a laundry detergent may cause skin irritations and may damage the fabric of the device.

THIS IS NOT A FIRE-PROOF DEVICE . DO NOT USE THE PRODUCT IN CLOSE PROXIMITY OF OPEN FIRE.

THE FINAL DECISION CONCERNING THE USE AND SELECTION OF THE DEVICE IS MADE BY A SPECIALISED PHYSICIAN